TELETRANSETT 915LR

ENGLISH: Receiver for wireless reception from inductive loops... page 2

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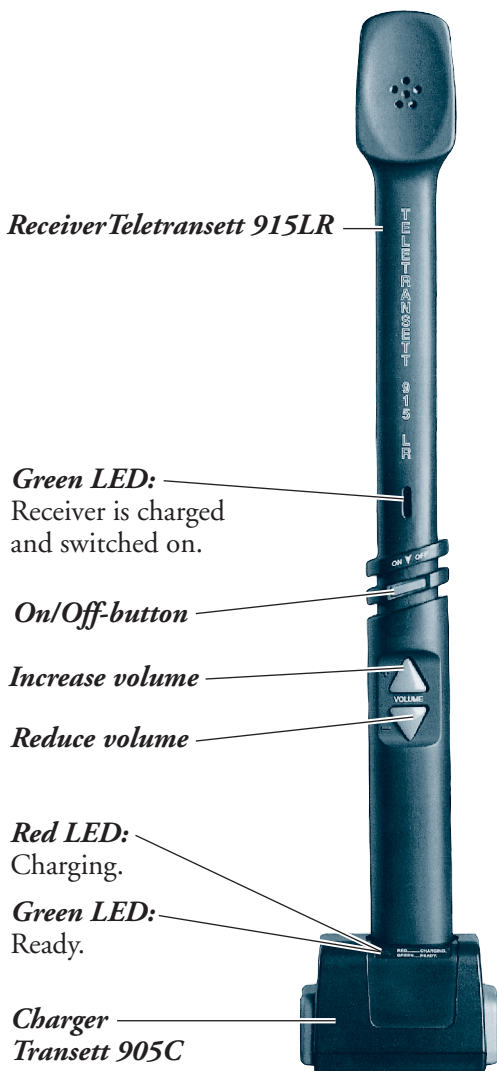
Receiver for reception from inductive loops

Teletransett 915LR is a receiver with built-in amplifier and speaker for wireless reception from inductive loops.

Teletransett 915LR has rechargeable NiMH batteries which are automatically charged when Teletransett 915LR is placed in the Transsett 905C charger. After an 80-minute quick recharge the receiver will operate for 10 hours.


Charging

Teletransett 915LR is charged by placing it in the charger Transsett 905C. Up to five chargers can be powered by one battery eliminator. **NB: Only use battery eliminator Transsett 905PS5 (Art.No: 415 06 03).**


A red light on the charger indicates that the receiver is charging. When the receiver is fully charged the light turns green and the charger switches to trickle charge mode. Teletransett 915LR is now ready for use and has an operating time of approximately 10 hours.

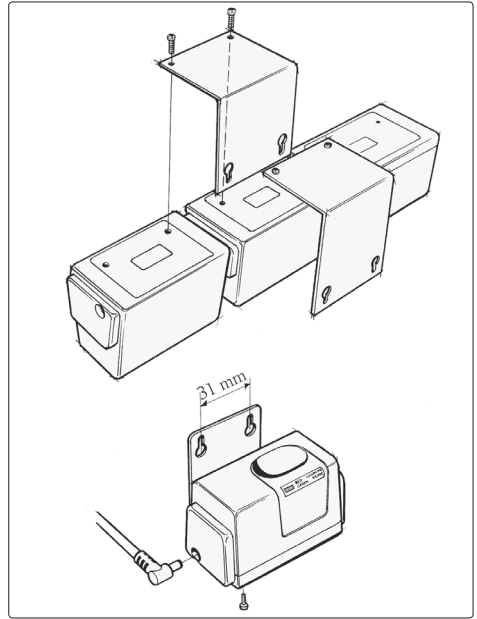The charger can be placed on a shelf or table, or mounted on a wall. A wall bracket is included with each charger.

Suspension

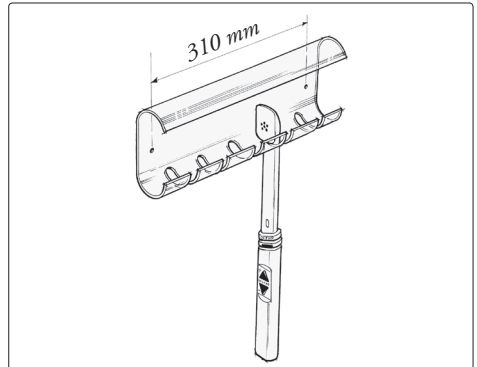A shelf for five Teletransett 915LR receivers is available as an accessory.

Troubleshooting


| Type of fault | Cause | Action |
|-------------------------------------|--|---|
| No/weak sound. | 1. Teletransett is not switched on. 2. The volume is set too low. | 1. Press the On/Off button so that the green LED comes on. 2. Adjust the volume with the volume buttons. |
| Teletransett cannot be switched on. | The battery is flat. | Charge Teletransett (see above). |
| Teletransett cannot be charged. | There is no power to the battery charger. | Check that the power supply unit is connected to a 220 volt socket and to the charger. |

Bass reduction

If there is too much bass, a bass reduction circuit can be switched on.

This can be done by opening the back cover and setting both switches to ON.

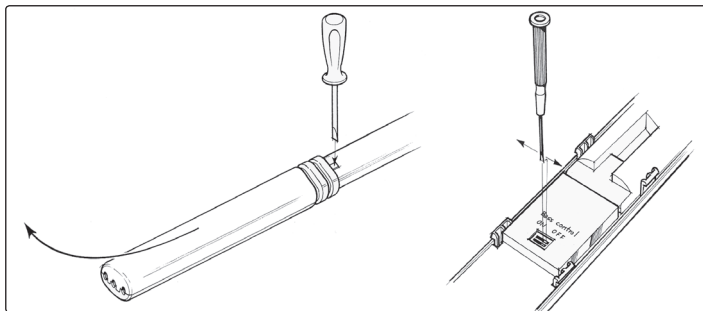As default setting, the bass reduction is switched off.

Maintenance

The Teletransett 915LR and Transett 905C need a minimum of maintenance. Wipe it occasionally with a moist cloth. Do not use any detergents or solvents. The lifetime of the battery is approximately five years in normal use. A new battery pack can be ordered from your supplier.

General Precautions

- Different types of radiation, e.g. from NMR or CT scanners, may damage the Assistive Listening Device. Therefore, do not wear the device during these or other corresponding scanning procedures. Other types of radiation (burglary alarms, room surveillance systems, radio equipment, mobile telephones, etc.) will not damage the device. They could, however, momentarily affect the sound quality or create strange sounds from the device.
- Warning: External devices connected to the product must be safe according to the requirements of EN 60601-1-1 or EN 60065, as appropriate.

Repairs

If your Assistive Listening Device malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your Assistive Listening Device requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

Accessories and ordering numbers

| | |
|---|-------------|
| Teletransett 915LR | 415 06 02 |
| Charger, Transett 905C | 415 06 04 |
| PS, Transett 905 PS5 (battery eliminator) | 415 06 03 |
| Shelf for five Teletransett 915LR | 414 05 07 |
| Quick guide in A5, with wall frame | 414 05 09 |
| Battery pack NiMH, 2.4 V | 874 582 600 |

Technical data

| | |
|--------------------------------|--|
| Max acoustic output level: | 125 dB SPL. |
| Output level when switched on: | 85 dB at 10 mA/m. |
| Frequency range: | 500 Hz-5 kHz, -3 dB. |
| Switchable bass cutoff: | Break point at 650 Hz, slope more than 12 dB/octave. |
| Operating time: | > 10 hours. |
| Battery type: | Battery pack with 2 R03 (AAA) nickel metal hydrid batteries. |
| Weight: | 185 g. |

CE Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to AB Transistor Sweden.

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