Companion (Favorit)

Order numbers:
180 83 100 (silver)
180 83 000 (black)

ENGLISH: Portable and user-friendly amplifier page 3

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Portable and user-friendly amplifier

Companion is a versatile conversation amplifier which operates according to the same principles as most standard hearing instruments. It is equally suitable for people with and without a hearing instrument. Wearers of hearing instruments listen via a neck loop and those without a hearing instrument listen via headphones. Companion's size and easy operation makes it convenient to take along with you and use in many different situations.


Overview

- Turn the volume knob to switch Companion on/off and adjust the sound level.
- Red LED = on.
- Built-in microphone.
- Switch between M (microphone) and T position (telecoil).
- Input for external microphone.
- Socket for headphones or neck loop.

Connection and starting


1. If you are using an external microphone, connect it to the Mic input.
2. Set the switch to the M position.
In public premises with an inductive loop the switch should be in the T position (see below).
3. Turn the volume knob to switch Companion on. Adjust the volume to **minimum** (see diagram).
4. Connect the headphones or neck loop to the headphone socket.
5. **Carefully** adjust the volume to a comfortable level. Hearing can be damaged if the sound level is too high.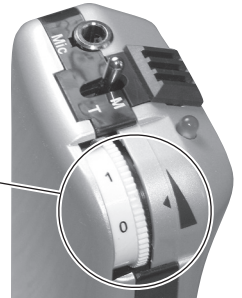


Public premises with an inductive loop

Companion has an integrated telecoil which receives signals sent on an inductive loop in public premises, e.g. churches. Set the switch to the T position. Hold Companion vertical to obtain the best sound quality.

If you are using a hearing instrument, switch it over to the T position. You do not need Companion in that case.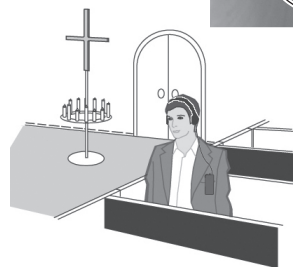


Changing batteries

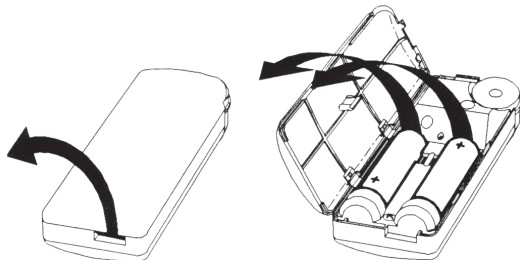NOTE: When you insert new batteries, press the batteries down towards the spring.

Controls under the battery cover

Under the cover there are two potentiometers. These control the gain (G) and bass cut (H). The potentiometers are preset at the factory. Adjustments may only be made to them by technically trained personnel.

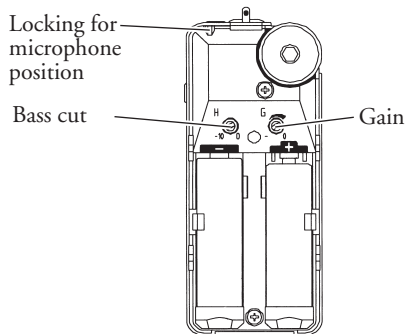Accessories

Mic, directional, Companion (Order No: 200 60 12): _____
Designed to reduce background noise.

Mic, Lapel 1.2 m, 3.5 mm (Order No: 411 03 15): _____
Attach to the person you are talking to.

Mic, FXT 4 m, 3.5 mm (Order No: 200 60 13): _____
Attach it to the TV or radio loudspeaker.

Neck Loop, 2-channel (Order No. 200 22 26): _____
For those wearing a hearing instrument.

Headphones, 3.5 mm Companion (Order No: 200 60 20): _____
Included. For those not using a hearing instrument.

Headphones, in-the-ear (Order No: 200 60 10): _____
Alternative to headphones.

Troubleshooting

Type of fault	Action
No sound is coming from Companion.	<ol style="list-style-type: none"> 1. Check that the batteries are OK. 2. Check that the connector is inserted in <i>the headphone socket</i>. 3. Check that the M/T switch is in the correct position.
There is a screeching sound from Companion.	The distance between the headphones and Companion is too short so that there is feedback. Increase the distance or reduce the volume.

Cleaning and maintenance

Use a damp cloth to wipe it on the outside. Never use cleaning products or solvents. Refer to the technical data below for the battery service life. Refer to page 4 for how to change the battery.

Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interests of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General precautions

- Hearing can be damaged if the sound level is too high. Follow the instructions for connection and starting, page 3.
- Different types of radiation, e.g. from NMR or CT scanners, may damage the device. Therefore, do not wear the device during these or other corresponding scanning procedures. Other types of radiation (burglar alarms, room surveillance systems, radio equipment, mobile telephones, etc) will not damage the device. They could, however, momentarily affect the sound quality or create strange sounds from the device.
- Warning: External devices connected to the product must be safe according the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

Technical data

Max. acoustic amplification:	60 dB (can be damped 16 dB with gain control).
Max. acoustic output level:	125 dB SPL.
Frequency range:	200 Hz – 10 kHz.
Bass reduction:	-10 dB at 200 Hz.
Telecoil sensitivity:	93 dB SPL, 1 kHz, 10 mA/m.
Power consumption:	Closed circuit current 7 mA.
Service life:	Approx. 200 hours with headphones.
Type of battery:	2 off 1.5 V penlight (LR6/AA).
Dimensions:	105 x 100 x 43 mm.
Weight:	94 g (including batteries).

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AB Transistor Sweden