Companion (Favorit)

Speech Amplifier 18083100
Loop Listener 18083100-T

earstore.se  Transistor
More ways to hear

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Intended purpose

The intended purpose of Companion (Favorit) is to enhance speech intelligibility through amplification of an audio source or via a telecoil system.

Intended user

The intended user is a person with mild to severe hearing loss who needs sound amplification.

Intended user group

The intended user group consists of people of all ages with mild to severe hearing loss who need sound amplification.

Principle of operation

The Companion (Favorit) either amplifies speech and surrounding sound or receives an audio signal via the built in telecoil receiver. The sound is then transferred to the user via headphones.

Hereby AB Transistor Sweden (named "AB Transistor" below) declares that this product, within Europe, follows the essential requirements of the Medical Device Regulation EU 2017/745. Declaration of conformity can be obtained from AB Transistor.

Portable and user-friendly amplifier

Companion is a versatile conversation amplifier which operates according to the same principles as most standard hearing instruments. It is equally suitable for people with and without a hearing instrument. Wearers of hearing instruments listen via a neck loop and those without a hearing instrument listen via headphones. Companion's size and easy operation makes it convenient to take along with you and use in many different situations.

Overview

- Turn the volume knob to switch Companion on/off and adjust the sound level.
- Red LED = on.
- Built-in microphone.
- Switch between M (microphone) and T position (telecoil).
- Input for external microphone.
- Socket for headphones or neck loop.

Connection and starting

1. If you are using an external microphone, connect it to the Mic input.
2. Set the switch to the M position. In public premises with an inductive loop the switch should be in the T position (see below).
3. Turn the volume knob to switch Companion on. Adjust the volume to **minimum** (see diagram).
4. Connect the headphones or neck loop to the headphone socket.
5. **Carefully** adjust the volume to a comfortable level. Hearing can be damaged if the sound level is too high.

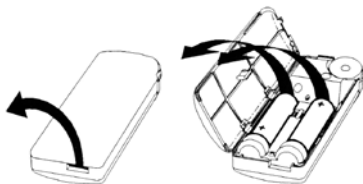
Public premises with an inductive loop

Companion has an integrated telecoil which receives signals sent on an inductive loop in public premises, e.g. churches. Set the switch to the T position. Hold Companion vertical to obtain the best sound quality.

If you are using a hearing instrument, switch it over to the T position. You do not need Companion in that case.


Changing batteries

NOTE: When you insert new batteries, press the batteries down towards the spring.

Controls under the battery cover

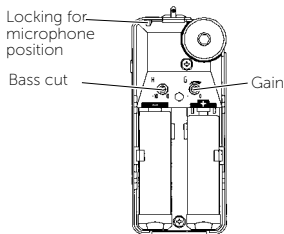Under the cover there are two potentiometers. These control the gain (G) and bass cut (H). The potentiometers are preset at the factory. Adjustments may only be made to them by technically trained personnel.

Accessories

Mic, Lapel 1.2 m, 3.5 mm (Order No: 411 03 15): _____
Attach to the person you are talking to.

Mic, FXT 4 m, 3.5 mm (Order No: 200 60 13): _____
Attach it to the TV or radio loudspeaker.

Neck Loop, 2-channel (Order No. 200 22 26): _____
For those wearing a hearing instrument.

Headphones, 3.5 mm Companion (Order No: 200 60 20): _____
Included. For those not using a hearing instrument.

Headphones, in-the-ear (Order No: 200 60 10): _____
Alternative to headphones.

Compliance information

This device is compliant with the following regulations and directives:

- Medical Device Regulation, MDR
- REACH Regulation
- RoHS Directive
- WEEE Directive

Hazard warnings

Failure to follow these safety instructions could damage the device or other property.

Keep this device out of reach of children under 3 years.

Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.

Protect cables from any potential source of damage.

Do not dismantle the device; Tampering with or dismantling the device will void warranty.

This device is designed for indoor use only. Do not expose the device to moisture.

Protect the device from shocks during storage and transport.

Do not make any changes or modifications to this device. Use only original AB Transistor accessories.

Information on product safety

Failure to follow these instructions could result in damage to the device and void the warranty.

Do not drop your device. Dropping onto a hard surface can damage it

Do not use the device in areas where electronic equipment is prohibited.

The device may only be repaired by AB Transistor or an authorized service provider.

If a serious incident occurs in relation to this device, contact the manufacturer. If you encounter other problems with your device, contact the point of purchase, or the manufacturer. Visit transistor.se for contact info.

Troubleshooting

Type of fault	Action
No sound is coming from Companion.	<ol style="list-style-type: none"> 1. Check that the batteries are OK. 2. Check that the connector is inserted in the headphone socket. 3. Check that the M/T switch is in the correct position.
There is a screeching sound from Companion.	The distance between the headphones and Companion is too short so that there is feedback. Increase the distance or reduce the volume.

Cleaning and maintenance

Use a damp cloth to wipe it on the outside. Never use cleaning products or solvents. Refer to the technical data below for the battery service life. Refer to page 4 for how to change the battery.

Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interests of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General precautions

- Hearing can be damaged if the sound level is too high. Follow the instructions for connection and starting, page 3.
- Different types of radiation, e.g. from NMR or CT scanners, may damage the device. Therefore, do not wear the device during these or other corresponding scanning procedures. Other types of radiation (burglar alarms, room surveillance systems, radio equipment, mobile telephones, etc) will not damage the device. They could, however, momentarily affect the sound quality or create strange sounds from the device.
- Warning: External devices connected to the product must be safe according to the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

Technical data

Max. acoustic amplification:	60 dB (can be damped 16 dB with gain control).
Max. acoustic output level:	125 dB SPL.
Frequency range:	200 Hz – 10 kHz.
Bass reduction:	-10 dB at 200 Hz.
Telecoil sensitivity:	93 dB SPL, 1 kHz, 10 mA/m.
Power consumption:	Closed circuit current 7 mA.
Service life:	Approx. 200 hours with headphones.
Type of battery:	2 off 1.5 V penlight (LR6/AA).
Dimensions:	100 x 46 x 25 mm.
Weight:	94 g (including batteries).

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